

THANK YOU FOR CHOOSING MY PERSONAL GENOME

You are a few steps away from unlocking the genetic code of your cancer. In order to proceed with the analysis, please do the following:

1. Complete the paperwork included
2. Provide a saliva sample
3. Make copies of your insurance cards (front and back)
4. Return these items in the envelope included

While you are waiting, MYPG will:

1. Obtain your tumor sample
2. Extract DNA from your saliva and tumor
3. Sequence the DNA of your tumor and saliva sample
4. Analyze your results
5. Compile your report and provide it to your doctor

Your CancerSELECT or CancerCOMPLETE report will include:

A list of possible treatment options specific for the genetic alterations found in your tumor. These treatment options may include:

- Therapies that are approved in your cancer type
- Therapies that are approved in other cancer types
- Clinical trials enrolling patients with your genetic alteration and tumor type

From the time MYPG receives your samples, the process takes three weeks for CancerSelect, and eight weeks for CancerComplete.

Don't hesitate to call us if you have questions about the process or the testing.

Personal Genome Diagnostics
support@personalgenome.com
1-888-571-2163

Learn more at personalgenome.com/patients-physicians
Watch video tutorial at personalgenome.com/how-to-order



SALIVA SAMPLE COLLECTION INSTRUCTIONS

Do not eat, drink, smoke or chew gum for at least 30 minutes before giving your sample. Failure to follow instructions exactly will delay your results and may result in additional charges



STEP 1

Fill tube with saliva to the fill line. It is important that you do not overfill.



STEP 2

Close funnel lid firmly, allowing buffer solution to mix with saliva. Unscrew and discard funnel top.



STEP 3

Use the small cap to close the tube tightly.



STEP 4

Gently shake the tube.



STEP 5

Write name, date of birth, and date of sample on provided label, then place label on tube.



STEP 6

Wrap the tube in the provided gauze.




STEP 7

Place the tube back into the Saliva Test Oragene-Dx box and firmly close.



STEP 8

Fill out the Payment and Insurance form. Include copies of the front and back of your insurance cards. Place all paperwork in the shipping envelope.

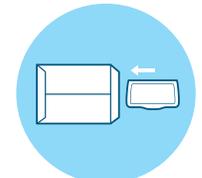

STEP 9

Return to MYPG by mailing through provided courier service.

Learn more at personalgenome.com/patients-physicians
Watch video tutorial at personalgenome.com/how-to-order

