



PlasmaSELECT™ 64

Blood Sample Information for Plasma

PLASMASELECT 64 BLOOD COLLECTION

Blood for PlasmaSELECT 64 testing must be collected in tubes provided in the PlasmaSELECT 64 kit. If additional kits are needed, please contact us at (888) 571-2163 or email customersupport@personalgenome.com

- Label tubes with 2 patient identifiers that will match the information on the Test Requisition Form (TRF)
- Collect 3 tubes of whole blood, filling tubes completely (10 ml in each tube)
- Mix tubes by gentle inversion 10 times

SHIPPING INSTRUCTIONS

1. Prepare Cold Pack 24 hours prior to plasma collection
2. Clinicians must complete the Test Requisitions Form to send alongside the sample
3. Patients must complete the Payment and Insurance Form that will be sent with the sample
4. Place labeled blood tube in Aqui Pak absorbent sleeve, then place into biohazard bag provided.
5. Place cold pak and biohazard bag with blood sample into Insultote foil pouch
6. Place foil pouch inside box
7. Place box in FedEx Clinical Pak (UN3373)
8. Affix FedEx return label to outside of FedEx Clinical Pak
9. Ship to Personal Genome Diagnostics by courier, so the sample is received within 3 days of blood draw.

TO BE INCLUDED

1. Completed Test Requisition Form
2. Three tubes of whole blood sample
3. Copies of the patient's insurance cards
4. Payment and Insurance information form (must be signed by patient)
5. Copy of recent Progress Note from his/her medical record

CONTACT

Sample Delivery Address:

Personal Genome Diagnostics
2809 Boston Street, Suite 503
Baltimore, MD 21224
Tel: (888) 571-2163

Sample Collection and Shipping:

lab@personalgenome.com
(443) 602-8833

Please send an email to lab@personalgenome.com to notify us that the sample has been sent.

Samples must be sent for arrival on Monday through Saturday.

LEARN MORE ABOUT PLASMASELECT 64 AT PERSONALGENOME.COM

FOR MORE DETAILS OR NOTIFICATION OF PRODUCT AVAILABILITY FOR ORDER, CONTACT CUSTOMERSUPPORT@PERSONALGENOME.COM

PERSONAL GENOME DIAGNOSTICS | 2809 BOSTON STREET, SUITE 503 | BALTIMORE, MARYLAND 21224 | TEL (888) 571-2163 | FAX (661) 212-5252

CS-002-V3